

Cochlear™ True Wireless™ Mini Microphones

Quick Guide for Cochlear implant users



Features of your Mini Microphone 2 / Mini Microphone 2+


Hear better over a distance and in noisy listening situations, such as:

- in meetings
- when travelling by car, bus or train
- dining at a restaurant
- participating in a class at the gym

How:

Give the Mini Microphone to the person you want to listen to and ask them to position it close to their mouth by placing it **vertically** on their clothing using the clip on the back. Then activate streaming on your sound processor (see instructions below).

Place your Mini Microphone 2+ on a table to listen to multiple speakers in a group / conference setting.

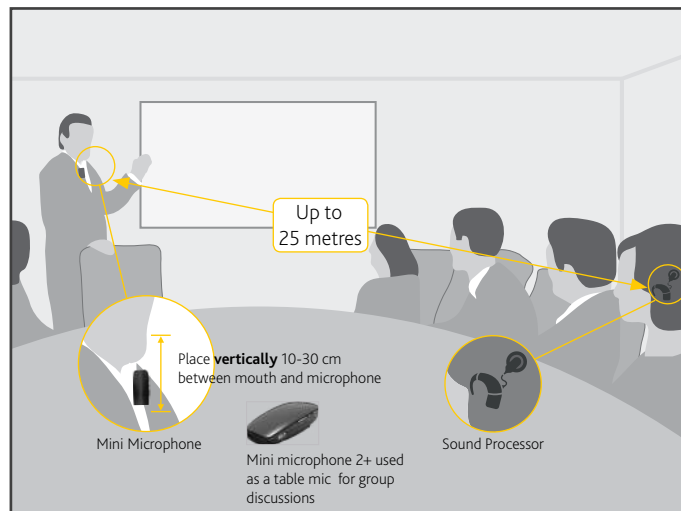

Product overview



Getting Started

Charging the battery

When using for the first time, allow your Mini Microphone to charge for at least 3 hours, even if the light indicates a full charge.

To charge your Mini Microphone:

1. Plug the Micro-USB charging cable into the Mini Microphone.
2. Plug the other end into a power outlet using the supplied adaptor.

While charging, the LED indicator lights red.

When fully charged (and still connected to the charger), the LED will turn solid red with green flashes every 2 seconds.

If battery level is low the Status indicator LED will blink yellow once every second.

On average, the Mini Microphone will operate for up to 10 hours between charges. Battery life may vary depending on use.


Pairing with your sound processor

- A** Turn off your sound processor(s)


Remove your sound processor(s).
Turn **OFF** your sound processor(s).

- B** Power on your Mini Microphone


Press and hold the ON/OFF button for 2 seconds until the LED turns green, then **release**.

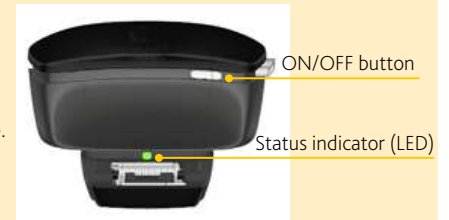

- C** Press the pairing button on the (A) Mini Microphone 2 or (B) Mini Microphone 2+ once using the tip of a pen or similar object.

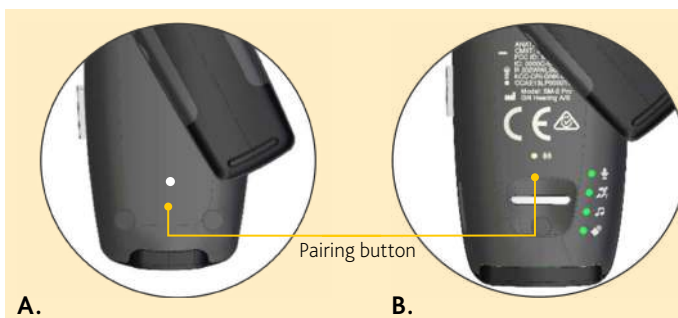

- D** The LED on top of the Mini Microphone will blink yellow once every 2 seconds for 20 seconds. While pairing mode is active (20 seconds), switch on your sound processor.

Turn **ON** your sound processor(s).
It briefly flashes a blue indicator light  to tell you pairing has been successful.

- E** You can pair up to three Cochlear wireless accessories with your sound processor(s).
To pair a Mini Microphone in channel 2 press the pairing button twice in step **C**. The LED will flash yellow twice every 2 seconds.
To pair a Mini Microphone in channel 3 press the pairing button 3 times in step **C**. The LED will flash yellow 3 times every 2 seconds.

Support

For full details about the Mini Microphone 2 and Mini Microphone 2+, please refer to the User Guide that was packaged with your Mini Microphone, and the True Wireless Accessories User Guide.

For full details about your Cochlear Sound Processor, please refer to its User Guide.


For further support and videos, visit the Cochlear website: www.cochlear.com

Using your Mini Microphone

Start streaming using your sound processor

A Turn on your sound processor and Mini Microphone.

Press and hold the button on your sound processor for 2 seconds, then release.

Your sound processor flashes a blue light  to indicate streaming.

NOTE: For processors with two buttons, use the top button.

B To select another channel or stop streaming.

Repeat Step A a second time to select Channel 2, and again for Channel 3.

To stop streaming, tap the button on your sound processor.

Start streaming using your Remote Assistant

Method 1: Turn on your sound processor and Mini Microphone. Turn on your Remote Assistant.

Method 2: Turn on your sound processor and Mini Microphone. Turn on your Remote Assistant.





Press and hold the Telecoil button for 2 seconds, then release.

Tip! Repeat this step a second time to select Channel 2, and again for Channel 3.

Audio will start streaming through the Mini Microphone.

To stop streaming, tap the Telecoil button.

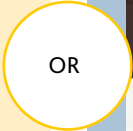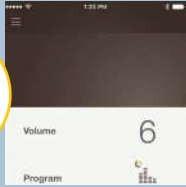From the Home screen, press the right arrow button until you see the Stream screen. Scroll down the list to the channel you want to use and press **OK**. When your audio device is turned on, audio will start streaming through the Mini Microphone. To stop streaming, select **Off** on the Stream selection screen.

Start streaming using your Remote Control

Turn on your sound processor and Mini Microphone. Turn on your Remote Control.

Start streaming using the Nucleus Smart App

Turn on your sound processor and Mini Microphone. Start the Nucleus Smart App.




Press and hold the Telecoil button for 2 seconds, then release.

Audio will start streaming through the Mini Microphone.

Repeat this step a second time to select Channel 2, and again for Channel 3.

To stop streaming, tap the Telecoil button.

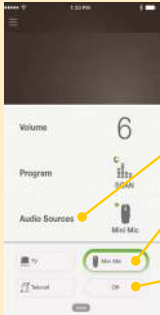

Tap Audio Sources.
Tap the Mini Mic icon.
Audio will start streaming through the Mini Microphone.
To stop streaming, tap the Off or X icon.